4 260159332780

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Armazenamento à chegada

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1 **Orange** 96 rxn *Yersinia* (lyophilized)

99.8% Synthetic oligonucleotides (< 0.01 picogram)

0.1% CAS 77-86-1 Tris (hydroxymethyl) aminomethane

0.1% CAS 60-00-4 Ethylenediamine tetraacetic acid (EDTA)

Do not freeze lyophilized reagents.

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1 **Black** Positive Control, Cp ~ 30

B. Intended Use

This product is intended to be used to diagnose an infection by detection of a bacterial genomic DNA in a nucleic acid extract obtained from watery stool samples from patients with signs of diarrhea. This assay has been developed for use in multiplex PCR and can be run combined with a control reaction or combined with other kits as specified in section F.

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V161122

C. Material Safety Data (MSDS)

EN: Product not hazardous, not toxic, not IATA-restricted, not from human, animal or plant origin.

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According to OSHA 29CFR1910.1200, Australia [NOHSC:1005,1008(1999)] and EU Directives 67/548/EC and 1999/45/EC products containing less than 1% of a component classified as hazardous or carcinogenic do not require a Material Safety Data Sheet (MSDS).

D. General Precautions and Warnings

EN: This product must be used by qualified personnel. Use the instructions as specified for the respective lot. Do not mix reagents from different lots. Do not use expired reagents. Dispose unused reagents.

The laboratory work-flow must conform to standard practices. Handle reagents at a different place than positive materials. Wear disposable protective gloves, laboratory coats and adequate eye protection. Sample materials and waste must be considered as potentially infectious. Reagent preparation, extraction and PCR should be performed in separated areas. Clean and disinfect work surfaces.

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TIB MOLBIOL Syntheselabor GmbH | Eresburgstr. 22-23 | D-12103 Berlin | Germany