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LightMix[®] Modular Sarbecovirus E-gene

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Total weight (incl. packaging) : 14 gram

B. Intended Use

This product is intended to be used to diagnose an infection by detection of the viral genome in a Total-NA nucleic acid extract obtained from respiratory tract specimen from patients with significant respiratory symptoms. This assay has been developed for use in multiplex PCR and can be run combined with a control reaction or combined with other kits as specified in section F.

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C. Material Safety Data (MSDS)

EN: Product not hazardous, not toxic, not IATA-restricted, not from human, animal or plant origin.

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According to OSHA 29CFR1910.1200, Australia [NOHSC:1005,1008(1999)] and EU Directive 67/548/EC products containing less than 1% of a component classified as hazardous or carcinogenic do not require a Material Safety Data Sheet (MSDS).

D. General Precautions and Warnings

EN: This product must be used by qualified personnel. Use the instructions as specified for the respective lot. Do not mix reagents from different lots. Do not use expired reagents. Dispose unused reagents.

The laboratory work-flow must conform to standard practices. Handle reagents at a different place than positive materials. Wear disposable protective gloves, laboratory coats and adequate eye protection. Sample materials and waste must be considered as potentially infectious (BSL2). Reagent preparation, extraction and PCR should be performed in separated areas. Clean and disinfect work surfaces.

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E. Procedures used for the Release of this Product (no instructions-for-use)

FR: Procédures utilisées pour la validation de ce produit (Ceci n'est pas un manuel d'instructions)
 ES: Procedimientos utilizados para el lanzamiento de éste producto (éste no es un manual)
 IT: Procedure utilizzate per il rilascio di questo prodotto (questo non è un manuale d'uso)
 PT: Procedimentos utilizados para a validação deste produto (não é um manual de instruções)
 DE: Verfahren für die Freigabe dieses Produktes (dies ist keine Bedienungsanleitung)

Detection Format 500 Channel

LightCycler® 480 Instrument:
 LightCycler® 480 II Instrument:
 cobas z 480 Analyzer (open channel):

Set Quant Factor 10, Max Integration time 1 sec

450-500
 440-488
 465-510 (can not be combined with FAM)

Program Step:	RT Step	Denaturation	Cycling			Cooling
Parameter						
Analysis Mode	None	None	Quantification mode			None
Cycles	1	1	45			1
Target [°C]	55	95	95	60	72	40
Hold [hh:mm:ss]	00:05:00	00:05:00	00:00:05	00:00:15	00:00:15	00:00:30
Ramp Rate [°C/s] 96	4.4	4.4	4.4	2.2	4.4	1.5
Ramp Rate [°C/s] 384	4.6	4.6	4.6	2.4	4.6	2.0
Acquisition Mode	None	None	None	Single	None	None

Table 1

E.1 Preparation of Parameter-Specific Reagents (PSR, 96 reactions):

Check for the colored pellet, then **add 50 µl** PCR-grade water, mix (vortex) and spin down.

► **Use 0.5 µl** reagent per 20 µl PCR reaction.

E.2 Preparation of the Positive Control

Add 160 µl RNase/DNase-free 10 mM Tris buffer pH 8 - 8.5 to the vial with the **black** cap. Mix by pipetting up and down 10 times. Pulse spin vial prior to opening. Store dissolved controls frozen.

► **Use 5 µl** positive control per 20 µl PCR reaction.

E.3. Single PCR Reaction Mix as used for the Release of this Product

In a cooled reaction tube, prepare the reaction mix for single reactions (left) or one plate (right):

Preparation of the Reaction Mix using Roche LightCycler® Multiplex RNA Virus Master		
One reaction	Component	100 reactions
10.4µl	Water , PCR-grade (colorless cap, provided with the Roche Master kit)	1040 µl
0.5 µl	Reagent mix (parameter specific reagents containing primers and probes)	50 µl
--	Control Reaction and additional assays (Multiplex PCR) – substitute water.	--
4.0 µl	Roche Master (see Roche manual)	400 µl
0.1 µl	RT Enzyme (see Roche manual)	10 µl
15.0 µl	Volume of Reaction Mix	1.500 µl

Table 2

Mix gently, spin down and **transfer 15 µl** per well.

Add 5 µl of sample or control DNA to each well for a final reaction volume of 20 µl. Seal plate and centrifuge.

Start run

F. Multiplex PCR

EN: This assay can be combined with other assays up to 6plex as depicted below :

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Multiplex PCR and Instrument Compatibility

Color Compensation 40-0320

500	530	580	610	640	660
Sarbeco					
Sarbeco	InfA	InfB	RSV-A/B	PicornaV	EAV

480 II	z 480	LC96	LC2.0	Nano
X	X	X		
X				

Table 4

G. Version History

EN: Red notes mark changes in procedures, blue change of components

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Release version

2020-02-04

Note. EU / German Export Restrictions for this product (Dual Use Bioweapon Detection). End-user-certificate may be required. End user will be reported to the National Authorities

Certificate of Analysis (CoA)							
FR: Certificat d'Analyse; ES: Certificado de análisis; IT: Certificato di analisi PT: Certificado de análise; DE: Analysenzertifikat							
Lot n° 4830							
Expiry :							
Dilution	1E6	1E5	1E4	PC	1E2	Cut off	passed
Cp range	18-21	21-23	24-26	28-30	30-33		
Measured Signal level						80-140	✓
Measured							✓
Negatives	20/20						✓
Notes: Cp (crossing point) values collected with pDNA in a single target PCR. Cp values may vary from instrument to instrument by up to 2 cycles, while the interval between two dilution steps is constant (Δ Cp). In multiplex PCR Cp values are delayed. The cut-off is an interval based on the Cp value for the positive control; the cut-off in the CoA is a recommendation and must be set by the user. Fluorescence signal levels depend on instrument settings. Reported values are related to one reference instrument of the manufacturer. cobas z480 Analyzer signal levels are approx 50% compared to LC480 II results (more narrow filter bandwidth).							
QC Acceptance Date:				YYYY-MM-DD			
We, the undersigned, certify that the product designated above has been obtained in accordance with the rules of production and quality control.							
Name(s) :							

TIB MOLBIOL Syntheselabor GmbH | Eresburgstr. 22-23 | D-12103 Berlin | Germany
Tel. +49 30 78 79 94 55 | FAX +49 78 79 94 99 | dna@tib-molbiol.de | WWW.TIB-MOLBIOL.COM
Geschäftsführer (CEO): Olfert Landt | Register HRB 93163 B | Registergericht Berlin Charlottenburg

Distributed by Roche -Instructions see: <http://elabdoc.roche.com>