4260159332988

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Armazenamento à chegada

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#### 1 **red** 96 rxn *Dientamoeba* (lyophilized)

99.8% Synthetic oligonucleotides (< 0.01 picogram)

0.1% CAS 77-86-1 Tris (hydroxymethyl) aminomethane

0.1% CAS 60-00-4 Ethylenediamine tetraacetic acid (EDTA)

#### Do **not** freeze lyophilized reagents.

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#### 1 **Black** Positive Control, Cp ~ 30

### B. Intended Use

**This product is intended to be used to diagnose an infection by detection of a parasitai genomic DNA in a nucleic acid extract obtained from watery stool samples from patients with signs of diarrhea. This assay has been developed for use in multiplex PCR and can be run combined with a control reaction or combined with other kits as specified in section F.**

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IT: Paese d'origine; PT: País de origem

## C. Material Safety Data (MSDS)

**EN: Product not hazardous, not toxic, not IATA-restricted, not from human, animal or plant origin.**

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According to OSHA 29CFR1910.1200, Australia [NOHSC:1005,1008(1999)] and EU Directives 67/548/EC and 1999/45/EC products containing less than 1% of a component classified as hazardous or carcinogenic do not require a Material Safety Data Sheet (MSDS).

## D. General Precautions and Warnings

EN: This product must be used by qualified personnel. Use the instructions as specified for the respective lot. Do not mix reagents from different lots. Do not use expired reagents. Dispose unused reagents.

The laboratory work-flow must conform to standard practices. Handle reagents at a different place than positive materials. Wear disposable protective gloves, laboratory coats and adequate eye protection. Sample materials and waste must be considered as potentially infectious. Reagent preparation, extraction and PCR should be performed in separated areas. Clean and disinfect work surfaces.

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## E. Procedures used for the Release of this Product (no instructions-for-use)

FR: Procédures utilisées pour la validation de ce produit (Ceci n'est pas un manuel d'instructions)  
 ES: Procedimientos utilizados para el lanzamiento de éste producto (éste no es un manual)  
 IT: Procedure utilizzate per il rilascio di questo prodotto (questo non è un manuale d'uso)  
 PT: Procedimentos utilizados para a validação deste produto (não é um manual de instruções)  
 DE: Verfahren für die Freigabe dieses Produktes (dies ist keine Bedienungsanleitung)

**Detection Format 580 Channel** **Set Quant Factor 10, Max Integration time 1 sec**  
 LightCycler® 480 Instrument: 523-568  
 LightCycler® 480 II Instrument: 533-580  
 cobas z 480 analyzer (open channel): 540-580

| Program Step:               | RT Step* | Denaturation | Cycling             |          |          | Cooling  |
|-----------------------------|----------|--------------|---------------------|----------|----------|----------|
| <b>Parameter</b>            |          |              |                     |          |          |          |
| Analysis Mode               | None     | None         | Quantification mode |          |          | None     |
| Cycles                      | 1        | 1            | 45                  |          |          | 1        |
| Target [°C]                 | 55       | 95           | 95                  | 60       | 72       | 40       |
| Hold [hh:mm:ss]             | 00:05:00 | 00:05:00     | 00:00:05            | 00:00:15 | 00:00:15 | 00:00:30 |
| Ramp Rate [°C/s] <b>96</b>  | 4.4      | 4.4          | 4.4                 | 2.2      | 4.4      | 1.5      |
| Ramp Rate [°C/s] <b>384</b> | 4.6      | 4.6          | 4.6                 | 2.4      | 4.6      | 2.0      |
| Acquisition Mode            | None     | None         | None                | Single   | None     | None     |

\* optional to combine with 1-Step RT-PCR

Table 1

### E.1 Preparation of Parameter-Specific Reagents (PSR, 96 reactions):

Add 50 µl PCR-grade water to each reagent vial, mix the solution (vortex) and spin down.

► Use 0.5 µl reagent per 20 µl PCR reaction.

### E.3 Preparation of the Positive Control

Add 160 µl RNase/DNase-free Tris buffer or water to the vial with the **black** cap. Use of buffer pH 8-8.5 increases the stability. Mix by pipetting up and down 10 times. If vortexing spin down to collect drops.

► Use 5 µl positive control (≈ 1,000 copies) per 20 µl PCR reaction.

### E.4. Single PCR Reaction Mix as used for the Release of this Product

In a cooled reaction tube, prepare the reaction mix for single reactions (left) or one plate (right):

| Preparation of the Reaction Mix using Roche LightCycler® Multiplex DNA Master |                                                                                |                 |
|-------------------------------------------------------------------------------|--------------------------------------------------------------------------------|-----------------|
| One reaction                                                                  | Component                                                                      | 100 reactions   |
| 10.5 µl                                                                       | <b>Water</b> , PCR-grade (colorless cap, provided with the Roche Master kit)   | 1050 µl         |
| 0.5 µl                                                                        | <b>Reagent</b> mix (parameter specific reagents containing primers and probes) | 50 µl           |
| --                                                                            | Control Reaction and additional assays (Multiplex PCR) – substitute water.     | --              |
| 4.0 µl                                                                        | <b>Roche Master</b> (see Roche manual)                                         | 400 µl          |
| <b>15.0 µl</b>                                                                | <b>Volume of Reaction Mix</b>                                                  | <b>1.500 µl</b> |

Table 2

Mix gently, spin down and transfer 15 µl per well.

Add 5 µl of sample or control DNA to each well for a final reaction volume of 20 µl. Seal plate and centrifuge.

**Start run**

## F. Multiplex PCR Gastro Parasites

**EN: This assay can be combined with other assays up to 6plex as depicted below :**

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### Multiplex PCR and Instrument Compatibility

Color Compensation 40-0320

| 500           | 530     | 580         | 610             | 640          | 660  |
|---------------|---------|-------------|-----------------|--------------|------|
| E.histolytica | Giardia | Dientamoeba | Cryptosporidium | Blastocystis | PhHV |

| 480 II | z 480 | LC96 | LC2.0 | Nano |
|--------|-------|------|-------|------|
| X      | X     | X    |       |      |
| X      |       |      |       |      |

Table 4

## G. Version History

**EN: Red notes mark changes in procedures, blue change of components**

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Distributed by Roche - [www.lifescience.roche.com](http://www.lifescience.roche.com)

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**TIB MOLBIOL** Syntheselabor GmbH | Eresburgstr. 22-23 | D-12103 Berlin | Germany