4260159335190

*In-vitro* diagnostics reagent. For *in-vitro* use only.

### Technical Information and Certificate of Analysis

Informations techniques et certificat d'analyse  
 Información técnica y certificado de análisis  
 Informazioni tecniche e certificato di analisi  
 Informação técnica e certificado de análise  
 Informacja techniczna oraz Certyfikat analizy  
 Informații tehnice și certificat de analiză  
 Specifikacije in Certifikat analize  
 Τεχνικές πληροφορίες και πιστοποιητικό ανάλυσης

Technische Informationen und Analysezertifikat  
 Technische informatie en Certificate of Analysis  
 Teknisk informasjon og Certificate of Analysis  
 Teknisk informasjon og analysesertifikat  
 Teknisk information och analyscertifikat  
 Technické informácie a Certifikát analýzy  
 Tekniset tiedot ja Certificate of Analysis  
 A dokumentáció és Analitikai tanúsítvány

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#### Do not freeze lyophilized reagents.

Ne pas congeler les réactifs lyophilisés.  
 No congelar los reactivos liofilizados.  
 Non congelare i reagenti liofilizzati.  
 Não congelar os reagentes liofilizados  
 Lyophilisierte Reagenzien nicht Einfrieren.

#### 1 **Black** Positive Control, Cp ~ 30

Total weight (incl. packaging) : 18 gram

### B. Intended Use

**This product is intended to be used to diagnose an infection by detection of the resistance gene in a nucleic acid extract obtained from rectal swabs or fecals samples. This assay has been developed for use in multiplex PCR and can be run combined with a control reaction or combined with other kits as specified in section F.**

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V180101

## C. Material Safety Data (MSDS)

**EN: Product not hazardous, not toxic, not IATA-restricted, not from human, animal or plant origin.**

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According to OSHA 29CFR1910.1200, Australia [NOHSC:1005,1008(1999)] and EU Directives 67/548/EC and 1999/45/EC products containing less than 1% of a component classified as hazardous or carcinogenic do not require a Material Safety Data Sheet (MSDS).

## D. General Precautions and Warnings

EN: This product must be used by qualified personnel. Use the instructions as specified for the respective lot. Do not mix reagents from different lots. Do not use expired reagents. Dispose unused reagents.

The laboratory work-flow must conform to standard practices. Handle reagents at a different place than positive materials. Wear disposable protective gloves, laboratory coats and adequate eye protection. Sample materials and waste must be considered as potentially infectious. Reagent preparation, extraction and PCR should be performed in separated areas. Clean and disinfect work surfaces.

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## E. Procedures used for the Release of this Product (no instructions-for-use)

FR: Procédures utilisées pour la validation de ce produit (Ceci n'est pas un manuel d'instructions)  
 ES: Procedimientos utilizados para el lanzamiento de éste producto (éste no es un manual)  
 IT: Procedure utilizzate per il rilascio di questo prodotto (questo non è un manuale d'uso)  
 PT: Procedimentos utilizados para a validação deste produto (não é um manual de instruções)  
 DE: Verfahren für die Freigabe dieses Produktes (dies ist keine Bedienungsanleitung)

**Detection Format 640 Channel** **Set Quant Factor 10, Max Integration time 3 sec**  
 LightCycler® 480 Instrument: 558-640  
 LightCycler® 480 II Instrument: 533-640  
 cobas z 480 Analyzer (open channel): 540-645

Program Step:	RT Step*	Denaturation	Cycling			Cooling
<b>Parameter</b>						
Analysis Mode	None	None	Quantification mode			None
Cycles	1	1	45			1
Target [°C]	55	95	95	60	72	40
Hold [hh:mm:ss]	00:05:00	00:05:00	00:00:05	00:00:15	00:00:15	00:00:30
Ramp Rate [°C/s] <b>96</b>	4.4	4.4	4.4	2.2	4.4	1.5
Ramp Rate [°C/s] <b>384</b>	4.6	4.6	4.6	2.4	4.6	2.0
Acquisition Mode	None	None	None	Single	None	None

\* optional to combine with 1-Step RT-PCR

Table 1

### E.1 Preparation of Parameter-Specific Reagents (PSR, 96 reactions):

Add 50 µl PCR-grade water to each reagent vial, mix the solution (vortex) and spin down.

► Use 0.5 µl reagent per 20 µl PCR reaction.

### E.3 Preparation of the Positive Control

Add 160 µl RNase/DNase-free Tris buffer or water to the vial with the **black** cap. Use of buffer pH 8-8.5 increases the stability. Mix by pipetting up and down 10 times. If vortexing spin down to collect drops.

► Use 5 µl positive control (≈ 1,000 copies) per 20 µl PCR reaction.

### E.4. Single PCR Reaction Mix as used for the Release of this Product

In a cooled reaction tube, prepare the reaction mix for single reactions (left) or one plate (right):

Preparation of the Reaction Mix using Roche LightCycler® Multiplex DNA Master		
One reaction	Component	100 reactions
10.5 µl	<b>Water</b> , PCR-grade (colorless cap, provided with the Roche Master kit)	1050 µl
0.5 µl	<b>Reagent</b> mix (parameter specific reagents containing primers and probes)	50 µl
--	Control Reaction and additional assays (Multiplex PCR) – substitute water.	--
4.0 µl	<b>Roche Master</b> (see Roche manual)	400 µl
<b>15.0 µl</b>	<b>Volume of Reaction Mix</b>	<b>1.500 µl</b>

Table 2

Mix gently, spin down and transfer 15 µl per well.

Add 5 µl of sample or control DNA to each well for a final reaction volume of 20 µl. Seal plate and centrifuge.

**Start run**

## F. Multiplex PCR Carbapenemase

EN: This assay can be combined with other assays up to 6plex as depicted below :

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### Multiplex PCR and Instrument Compatibility

Color Compensation 40-0320

500	530	580	610	640	660
VIM	NDM	OXA48	KPC	GES	
VIM	NDM	OXA48+23	KPC	IMP+GES	PhHV

480 II	z 480	LC96	LC2.0	Nano
X	X	X		
X				
X				

Table 4

## G. Version History

EN: Red notes mark changes in procedures, blue change of components

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Distributed by Roche - [www.lifescience.roche.com](http://www.lifescience.roche.com)

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TIB MOLBIOL Syntheselabor GmbH | Eresburgstr. 22-23 | D-12103 Berlin | Germany